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en - english

MediCult Vitrification Warming is for warming of vitrified human oocytes, cleavage stage embryos and blastocysts. This product is for ART treatment, whether the cause of infertility is male or female. The product should only be used by professionals trained in ART treatment.

Package
1224 Warming Medium (Vial 1)
1225 Dilution Medium 1 (Vial 2)

1226 Dilution Medium 2 (Vial 3)
1227 Washing Medium (Vial 4)

Pack size
1 x Vial 1 (2 ml)
1 x Vial 2 (2 ml)
1 x Vial 3 (2 ml)
2 x Vial 4 (2 x 2 ml)

Contains

Human albumin solution (HAS)
Gentamicin sulphate 10 µg/ml

Quality control testing
Sterility tested (Ph.Eur., USP)
pH tested (Ph.Eur., USP)
Endotoxin tested ≤ 0.5 EU/ml (Ph.Eur., USP)
HSA analysis (Ph.Eur., USP)
Mouse Embryo Assay (MEA) tested
Only Vial 4: Osmolality tested (Ph.Eur., USP)
Note: The results of each batch are stated on a Certificate of Analysis, which is available at www.origio.com.

Storage instructions and stability
The products are aseptically processed and supplied sterile. Store in original container at 2–8°C, protected from light. Do not freeze prior to use. Discard excess (unused) media following warming. The product is provided in vials intended for single use. When stored as directed by the manufacturer the product is stable until the expiry date shown on the label.

Precautions and warnings

Do not use the product if:

- Product packaging appears damaged or if the seal is broken.
- Expiry date has been exceeded.
- The product becomes discoloured, cloudy, turbid, or shows any evidence of microbial contamination.

Caution: The product is for single use and is not to be reused due to risk of contamination.

Caution: All blood products should be treated as potentially infectious. The source material used to manufacture this product was tested and found non-reactive for HBsAg and negative for Anti-HIV-1/-2, HIV-1, HBV, and HCV. Furthermore, source material has been tested for parvovirus B19 and found to be non-elevated. No known test methods can offer assurances that products derived from human blood will not transmit infectious agents.

Note: Please note the need for traceability of this product. In addition, national legal requirements for this field may exist in your country.

Note: Only to be used in combination with other devices intended for the particular purpose.

Note: Dispose of the device in accordance with local regulations for disposal of medical devices.

Instructions for use

- Pre-warm the Warming Medium to 37°C and Dilution Medium 1, Dilution Medium 2 and Washing Medium to room temperature for at least 30 minutes.
- Prepare a reservoir with enough liquid nitrogen (LN₂) to allow complete submersion of a goblet on a cryocane.
- Collect the cryocane and goblet containing the carrier device with vitrified oocytes or embryos/blastocysts from the storage container and quickly transfer them to the LN₂ reservoir. Make sure the carrier device remains submerged under LN₂.
- Mix the content of the individual vials by a few gentle inversions prior to use.
- In separate dishes, place 2 ml of Dilution Medium 1, Dilution Medium 2 and 2 x 2 ml Washing Medium, respectively.
- Place 2 ml of Warming Medium at 37°C in a pre-warmed dish just before opening the carrier device according to the manufacturer’s instruction for use.
- Quickly transfer the oocytes or embryos/ blastocysts into the Warming Medium, and leave them for 1–3 minutes (at this point, the cells are still shrunken).
- Using a suitable pipette and a minimum volume, transfer the oocytes or embryos/ blastocysts into Dilution Medium 1 at room temperature. Leave for 3 minutes (at this point, the cells will start to re-expand).
- In minimum volume, transfer the oocytes or embryos/blastocysts into Dilution Medium 2 and leave for 3 minutes (at this point, the cells will continue their re-expansion).
- In a minimum volume, transfer the

oocytes or embryos/blastocysts into the Washing Medium and leave for 3 minutes. Repeat the washing step by transferring the oocytes or embryos/ blastocysts to another dish containing the Washing Medium (at this point, the cells are fully re-expanded).

- Transfer the oocytes or embryos/ blastocysts into the preferred culture medium equilibrated according to the manufacturer’s instruction for use and allow them to rest in the incubator for minimum of 2 hours before visual inspection.

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Envase
1224 Warming Medium (Vial 1)
1225 Dilution Medium 1 (Vial 2)
1226 Dilution Medium 2 (Vial 3)
1227 Washing Medium (Vial 4)

Tamaño del envase
1 x Vial 1 (2 ml)
1 x Vial 2 (2 ml)
1 x Vial 3 (2 ml)
2 x Vial 4 (2 x 2 ml)

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